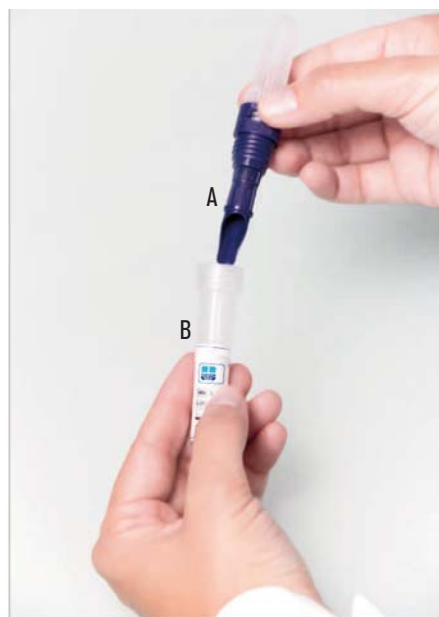
FIXATION AND CONCENTRATION OF FAECAL PARASITES “AQUISFEK”

New compact design of Aquisfek device, which allows an easy and convenient sample collection, a safe transportation with an easier handling system and a more effective filtration.

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SAMPLE COLLECTION.

EASY TO HANDLE BY THE PATIENT.

Separate the two parts (A and B) of the device and with the aid of the spoon built-in at the end of the filter, collect the sample.

Introduce the spoon with the sample back into the liquid and screw to close.

PROCESSING THE SAMPLES IN THE LABORATORY.

FAST AND EASY.

Once the sample has been homogenized, the whole device is centrifuged.

After centrifugation, discard the part A (spoon) and with the aid of a pipette, remove the supernatant in the tube (Part B).

The concentrated sample residue is transferred to a slide for a further microscopic observation.

TWO DIFFERENT FORMATS.

KIT 1. WITH FORMALIN / ACETATE. PARASITE COMMONL FIXER.

KIT 2. WITHOUT FORMALIN TO MINIMIZE THE RISK OF ENVIRONMENTAL CONTAMINATION.

KIT 1.

Part number

1800402 72 units supplied (24 units with 3 units in each bag) .

1800404 100 individual units supplied.

KIT 2. Without formalin.

Part number

1800400 72 units supplied (24 units with 3 units in each bag) .

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